# Basic / Profi

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EN The complete manual can be downloaded here:

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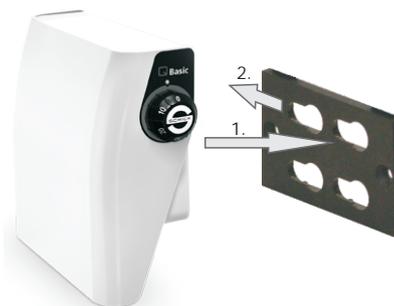
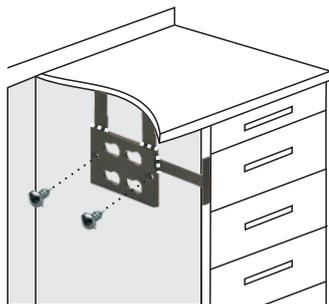
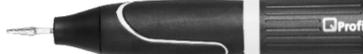
EN \*Connection of the signal transmitter for automatic activation of a workplace suction device: When using a Renfert Silent TS suction, the signal transmitter has to be switched to another program as described here: [www.schick-qube.de/support](http://www.schick-qube.de/support)

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## 1. Safety information

- 1.1 Ascertain that your mains supply coincides with the data on the rating plate.
- 1.2 **QBasic**/**QProfi** units are not suitable for the following applications:
  - In areas where there is a risk of explosion
  - for medical applications
  - for working on moist materials
- 1.3 Ensure that all regulatory requirements are observed during use:
  - always wear protective glasses
  - provide enough light at the working place
  - use dust suction
- 1.4 Under no circumstances should the motor hand-piece be cleaned with compressed air.
- 1.5 Before putting the handpiece down, always insert a rotary instrument or the pin, supplied with the unit, into chuck.

## Caution!

- When using rotary instruments, do not exceed the maximum speeds laid down by their manufacturer.
- When operation with left-hand rotation, the collet may loosen when there is a high load.
- Repairs and other technical procedures must only be carried out by suitable qualified personnel, authorized by SCHICK.
- SCHICK do not guarantee the **QBasic**/**QProfi** unit should it not have been used in accordance with the operating instructions.

## 2. Range of applications

The **QBasic**/**QProfi** unit is designed for universal use in dental laboratories when trimming crowns and bridges, respectively acrylic and light chrome cobalt dentures. With its speed range from 1000 - 40/50.000 rpm the SCHICK **QBasic**/**QProfi** unit allows to work all dental materials.

## Conditions of environment:

- interior 5° - 40° C.
- up to 2,000 meter over sea level
- Category of overvoltage: II
- Grade of pollution: 2

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Schick GmbH