Georg Schick Dental GmbH  
Lehenkreuzweg 12  
D-88433 Schemmerhofen  
Telefon 07356/9500-0  
Telefax 07356/9500-95  
E-Mail [info@schick-dental.de](mailto:info@schick-dental.de)  
Internet [www.schick-dental.de](http://www.schick-dental.de)

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Operating Instructions  
Schick C1 Junior

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We are pleased that you decided to buy a highly developed piece of equipment from SCHICK and would like to wish you every success when working with your new C1 motor unit.

We wrote these operating instructions to enable you to get accustomed to your new piece of equipment and to provide you with the correct operating and maintenance instructions.

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## Index

1. Summary C1-units
2. List of contents - accessories
3. Range of applications
4. General information  
**Caution !**
5. Setting up / operating the unit
6. Exchanging the rotary instruments
7. Replacing the chuck
8. Maintenance and care
9. Possible faults
10. Technical data
11. Declaration of conformity

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




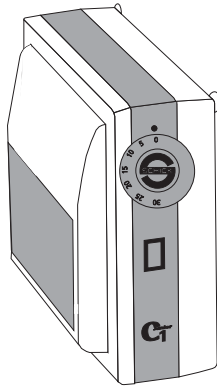
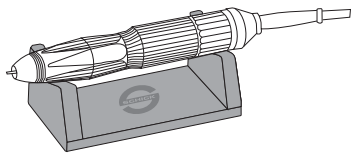
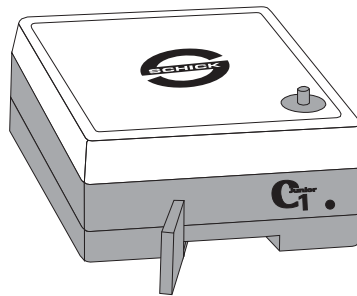
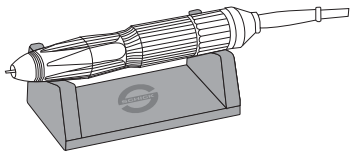
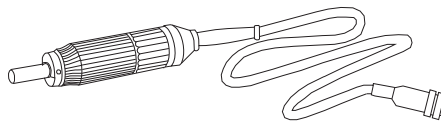
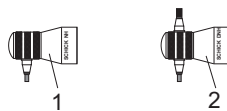
riveting hammer 1  
Burin 1  
twin riveting hammer 2  
double-burin 2

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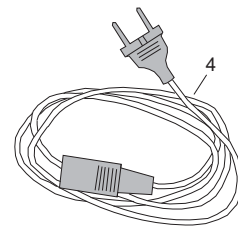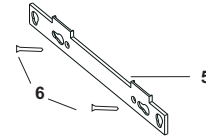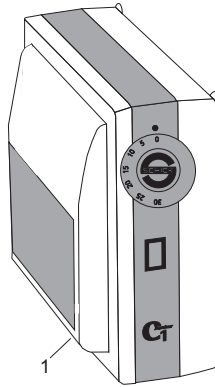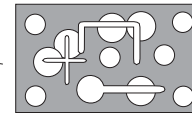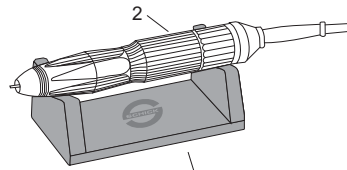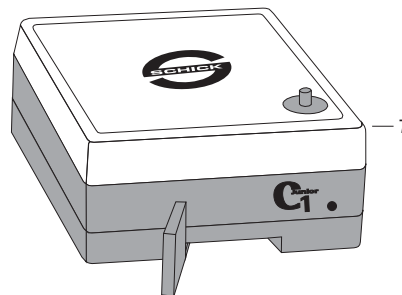
## 2. List of contents - accessories/Spare parts

		Art.-No.
C1/SK	motor unit complete	3305
C1/SK	knee controller 1	3335
C1/MH	motor-handpiece and cable 2	3300
HA	handpiece rest	6385
	with two keys for changing the chuck 3	
	mains cable 4	5361
	suspension strip 5	5614
	screws - 2 pieces - 6	3170
C1/SF	motor unit complete	3310
C1/SF	foot controller 7	3340
C1/MH	motor-handpiece and cable 2	3300
HA	handpiece rest	6385
	with 2 keys for changing the chuck 3	
	mains cable 4	5361
ET	Chucks:	
	Standard: Ø 2,35 mm	4114
	Optional: Ø 3,0 mm	4117
	Ø 3,175 mm	4332
	chuck key	4115
	counter stay wrench	6223
	motor cable cpl.	3550

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### 3. Range of applications

The C1 unit is designed for use in dental laboratories when trimming crowns and bridges, respectively light acrylic and chrome cobalt dentures.

The SCHICK - C1 motor unit is constructed (CPU controlled, robust, no commutator) so as to run smoothly, thus eliminating user-fatigue and making it economical to use.  
The processor controller constantly monitors the unit for overloading. Second thermal fuse, which is independent of the computer, provides further protection.

Conditions of environment:

- interior 5° - 40°
- up to 2,000 meter over sea level

Categorie of overvoltage: II  
Grade of pollution: 2

### 4. General Information

- 4.1 Ascertain that your mains supply coincides with the data in the rating plate.
- 4.2 C1-units are not suitable for the following applications:
  - in areas where there is a risk of explosion
  - on patients
  - for working on moist materials
  - for heavy working on acrylic and chrome cobalt materials
- 4.3 Ensure that all regulatory requirements are observed during use (always wear protective glasses).
- 4.4 **Under no circumstances should the motor handpiece be cleaned with compressed air.**
- 4.5 Before putting the handpiece down, always insert a rotary instrument or the pin, supplied with the unit, into the chuck.

#### Caution:

- When using rotary instruments, do not exceed the maximum speeds laid down by their manufacturer.
- Repairs and other technical procedures must only be carried out by suitable qualified personnel, authorized by SCHICK.
- Schick do not guarantee the C1 unit should it not have been used in accordance with the operating instructions.

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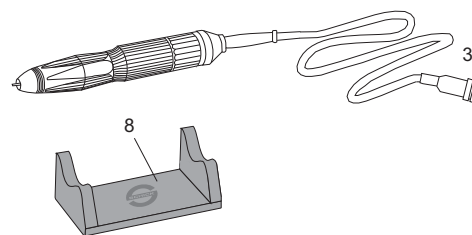
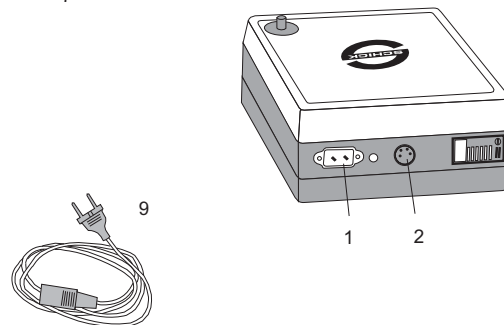
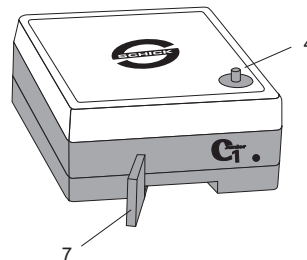
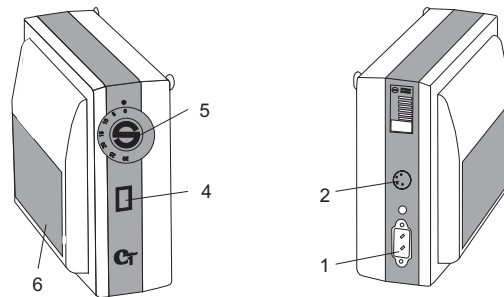
## 5. Setting up

### Operating the knee and foot controller versions

- 5.1 Ascertain that your mains supply coincides with the data on the rating plate.
- 5.2 Plug the mains cable **9** into the unit **1** and into a mains socket.
- 5.3 Plug the motor cable **3** into the socket **2** on the rear of the unit (quarter-turn catch).
- 5.4 Place the handpiece on the rest **8** supplied with it.
- 5.5 Press the mains switch **4** on the front of the unit (for the foot controller version - on the top). The control lamp will come on. The unit is switched off by pressing the mains switch **4** again.

### Applies to the knee controller version only

- 5.6 Set the speed controller **5** to the desired speed. The set speed can be reached right away by pressing the knee controller **6**.

### Applies to the foot controller version only

- 5.7 The speed can be varied progressively between 2.500 - 30.000 rpm by moving the pedal **7**.

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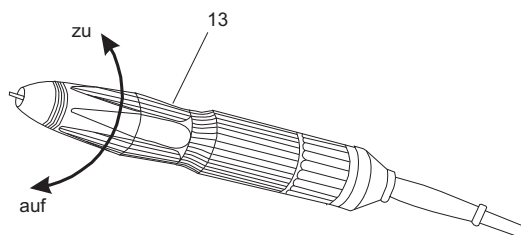
## 6. Exchanging the rotary instruments

The chuck is opened, respectively, closed by turning the outer sleeve 13 on the handpiece.

### Caution:

Only ever exchange the rotary instrument with the **motor switched off!**  
With regard to the precision and service-life of the chuck, an instrument must always be inserted into it - even when the handpiece is not in use.

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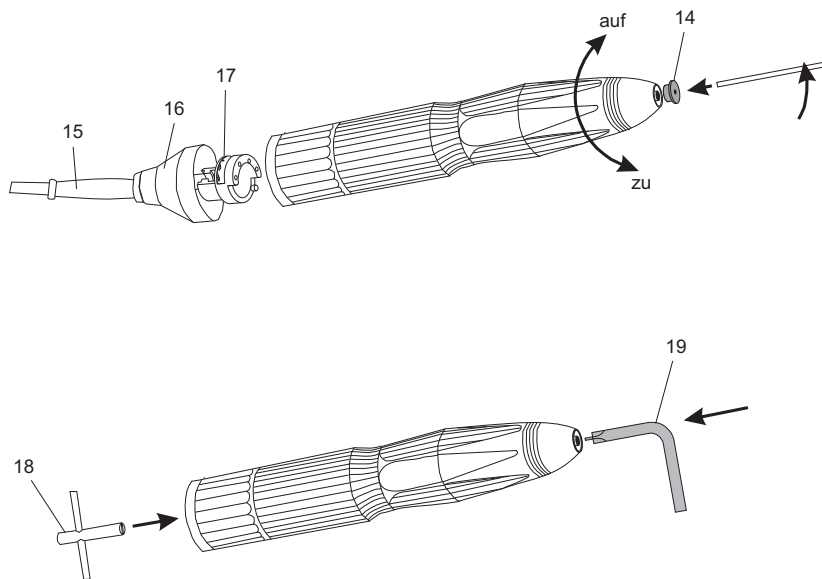
## 7. Replacing the chuck

- 7.1 Open the chuck and withdraw the rotary instrument.
- 7.2 Remove the covering cap 14 by using the shank of your rotary instrument or the pin supplied with the handpiece.
- 7.3 Remove the motor cable 15. To do so unscrew the cap 16 from the motor and loosen the cable by pulling it out of the plug-in seal 17. Please insure that the chuck is open.
- 7.4 Use a number 6223 wrench 18 to hold the motor end of the spindle. Insert chuck key number 4115 19 into the chuck. Hold the number 6223 wrench tightly and turn the chuck tool anti-clockwise to unscrew the chuck completely - the chuck has a right-handed thread!
- 7.5 Clean the chuck, grease it lightly and place it in the spindle. Use the tools to screw the chuck in - clockwise and as far as possible - as described in section 7.4, and **tighten it slightly**. Replace the plug-in seal 17 and screw the cap 16 back into place. Replace the covering cap 14

### Please note:

The tools 18 and 19 are to be found in the base of the handpiece rest!

If you intend to replace the standard 2,35 mm chuck with a 3.0 mm version, the protecting cap 14 must be replaced with a new one (article no. 6228).

When using a special handpiece for left hand the chuck is replaced accordingly, but with the left handed thread.

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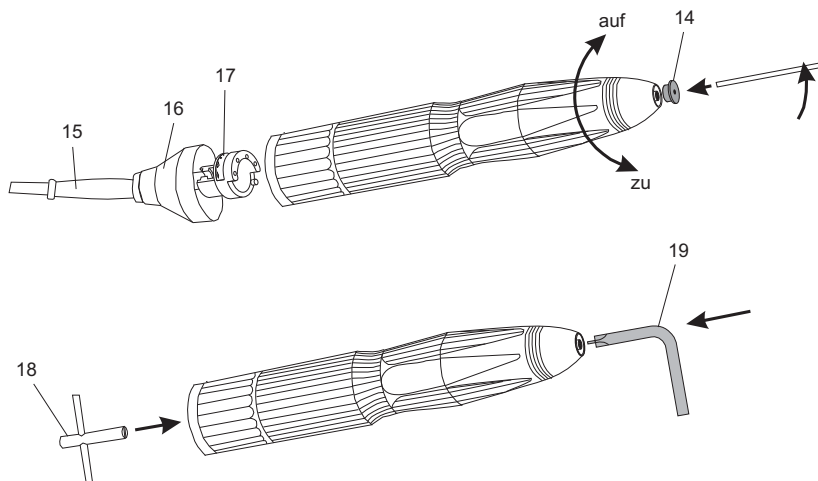
## 8. Care and maintenance

**Caution: Do not clean your C1 motorhandpiece with compressed air !**  
The chuck should be cleaned and re-greased once in a while, depending on how dirty it is.  
As C1 units have no commutators, carbon brushes or ventilation parts, no further maintenance is required.

## 9. Possible faults

Should the handpiece be overloaded, respectively, jammed, for safety reasons the unit will switch itself off.  
- Release the pedal switch, respectively knee switch and operate it again; the unit is ready for use immediately

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## 10. Technical Data

		C1/SK	C1/SF
Dimensions:	Width:	120mm	230mm
	Height:	230mm	110mm
	Depth:	245mm	245mm
Weight:		3,3 Kg	3,3Kg
	C1-MH	Length: 173mm Weight: 225g	
Power supply:		AC 220-240V	115V 100V
Nominal frequency:		50/60Hz	
Power input:		100W	
Type:		C1	
Handpiece model:		3xx.xxx	
Speed range:		2.500 - 30.000 rpm	
Safety class		II	

This unit complies with the current VDE (German association of electrical technicians) regulations concerning safety and suppression. The C1 is a group III unit according to Med. GV type B, and does not include an application component.

These operating instructions should be readily accessible and are best kept close to the unit itself.

We would like to take this opportunity to advise you that a proper repair service and suitable qualified personnel are required for such highly developed technical equipment.

SCHICK guarantees to carry out perfect repairs using original spare parts.

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Nous, GEORG SCHICK DENTAL GmbH