# GEORG SCHICK DENTAL GmbH

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## 1. Operating Instructions/all types

- 1.1 Check if you have the right type of plug.
- 1.2 Plug in the apparatus.
- 1.3 Place and screw in the motor plug into the insert at the front (model SK and model SF at the rear) of the apparatus.
- 1.4 Place the handpiece on the hanger provided.
- 1.5 Depress the button located at the front of the apparatus (model SF button at the upper side) - the control light will come on. Switch off by depressing the button.
- 1.6 The control unit can be delivered for use by hand, knee or foot. The table model can be supplied with a foot switch, a foot-control or a knee control according to your wishes.
  - 1.6.1 **Table model** - handcontrolled  
Put speed selection to the position chosen; the knob should be put to zero position after use.
  - 1.6.2 **Knee model**  
Program the maximum speed desired. With the knee switch the speed can be set infinitely variable up to the maximum speed programmed.
  - 1.6.3 **Foot model**  
Press button on the right top side of the unit to fix the desired maximum speed of 20.000 rpm or 40.000 rpm. If the maximum speed 20.000 rpm is chosen the lamp is shining green; otherwise (40.000 rpm) the lamp is shining red. With the foot switch the speed can be set infinitely variable up to the maximum speed fixed.
- 1.7 Table model  
If a controller or foot switch is used you must plug in and screw the plug at the rear of the apparatus.
  - 1.7.1 In case of utilization with a foot switch set the program on the desired figure. The apparatus will start by pressing the foot switch and runs then with the speed selected.
  - 1.7.2 To utilize with the foot- or knee control, program the maximum speed desired. By using foot-resp. knee-control the speed can be set infinitely variable up to the maximum speed programmed.

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## 2. Changing of Bur

2.1 By turning the body of the hand- piece, one can insert or retrieve the bur. When the green band is visible at the tip of the handpiece, the chuck is tight and the tool is ready for use.

**BEWARE: Switch off the motor before changing the bur.**

Please take care that the burs are used only up to the maximum speed stated by the manufacturers of these tools.

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## 3. Changing of Chuck

3.1 Open the chuck (see 2.1).

3.2 Unscrew the tip of the handpiece with the key provided or a coin. Insert the key in both planes of the shaft and pull out the chuck. Key ref. no. 5160.

3.3 Slightly grease the chuck, insert into shaft and screw **lightly** with the key and retrieve same.

3.4 After changing the chuck, screw the tip on to the handpiece.

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## Derangement

If the motor will not function after a correct starting procedure, re-start the operation from 1.5.

If the handpiece is overloaded or jamed, the apparatus will stop by security measure. Start again.

In case of overheating, the cut-out switch will operate, this is located at the rear of the apparatus. At normal tem- perature, press the stud and the appa- ratus will start again.

## Beware

This motor is **without carbon brushes**, therefore no servicing required. For the handpiece, clean from time to time the chuck and grease slightly (see 3.3).

## Replacement of printed circuit boards

1. Disconnect mains.
2. Remove screws from top cover (light grey coloured).
3. Remove the two screws, that hold the circuit boards together, the upper board (performance board) can be removed by pulling towards the rear (plug-in type).
4. Remove both braces and pull out the lower board (control board) towards the rear.
5. Re-install new circuit boards by inverting the above operation.

technical data:  
r.p.m.: 5.000 - 40.000  
watts: 200  
voltage: 110/220-240 ~  
Hz: 50/60

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