SCHICK

Dentalgeräte & Dentalinstrumente

Georg Schick Dental GmbH

Lehenkreuz 12 D-88433 Schemmerhofen
Telefon 07356/9500-0 Telefax 07356/9500-95

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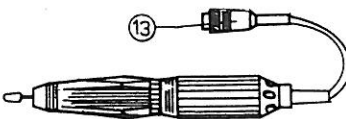
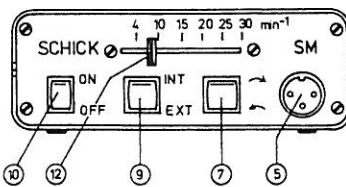
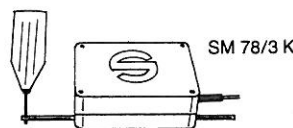
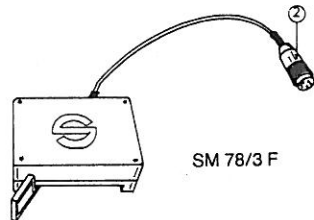
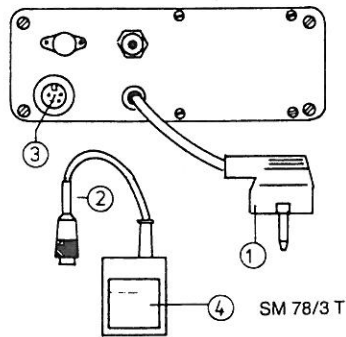
INSTRUCTIONS FOR USE for SCHICK MICRO-MOTOR

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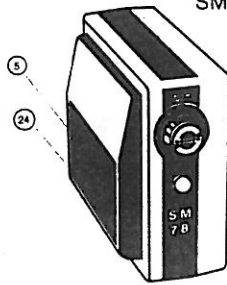
1. Putting into operation Type SM 78/3

- 1.1 Check if you have the right type of plug.
- 1.2 Plug in the power plug.
- 1.3 Connect and screw plug 13 into plug 5.
- 1.4 Connect and screw plug 2 into socket 3. (It is possible to connect to the socket the foot switch, and the foot or knee speed variator).
- 1.5 Place switch 9 according to the mark on position "EXT".
- 1.6 Place mains switch 10 on position "ON" (switch is shining).
- 1.7 Place switch 7 for sense of rotation on position "↻".
- 1.8 Put the handpiece on the holder.
- 1.9 Operate foot control 4. (motor is running)
- 1.10 Vary speed with cursor 12. When using foot or knee speed variator switching on and off the motor and continuous speed control from 4'000 to 30'000 min⁻¹ can be carried out by pressing the foot or knee lever.
- 1.11 If left hand sense of rotation is desired, place switch 7 on position "↻".
- 1.12 If switch 9 is placed on position "INT" the unit can be operated without foot switch and foot and knee speed variator. To switch-off-motor place button 10 on position "OFF".

Putting into operation Type SM 78/4K + 4F

- 1.1 Check if you have the right type of plug.
- 1.2 Plug in the apparatus.
- 1.3 Connect and screw plug 13 into plug 5.
- 1.4 Put the handpiece on the holder.

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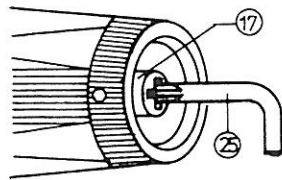
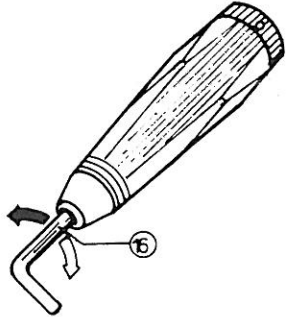
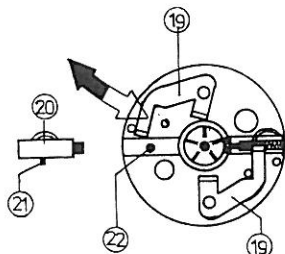
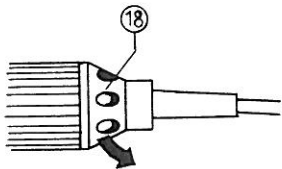
SM 78/4K

- 1.5 Depress the button located at the front of the apparatus (the control light will come on). Switch off by depressing the button again.
- 1.6.1 **variable speed**
place speed selector on position S. With the knee-switch the speed can be set infinitely variable from 4.000 - 30.000 min⁻¹.
- 1.6.2 **speed-selection**
program the speed desired. With the knee-switch the motor will be switched on or off.

SM 78/4F

- 1.5 The motor is switched on by operating the foot lever. With the foot lever the speed can be set infinitely variable from 4.000 - 30.000 min⁻¹.

2. Exchange of tools

- 2.1 Turn the handle sleeve nut 14 in the sense of direction sign ↗ to the stop.
- 2.2 Introduce tool in the chuck 15.
- 2.3 Turn the tightening nut 14 in the sense of direction sign ↘ to the stop.
Attention: tool change only when motor is stopped.

3. Change of the chuck

- 3.1 Open the chuck (see 2.1).
- 3.2 Turn the chuck 15 with the special key 16 with a sharp movement and unscrew it, in the sense of direction sign ↖.
- 3.3 Screw chuck 15 with the special key 16 in the sense of direction sign ↗ until feeling of resistance.
- 3.4 If after a long using period the chuck is blocked, remove the handpiece from the motor (see 5.1 to 5.3) in order to maintain the carrier 17 with the enclosed special key 25.

4. Change of carbon brushes

- 4.1 Unscrew cap 18 in direction of arrow ↖ and pull back. Disconnect cable plug.
- 4.2 Turn by finger withhold spring 19 in the sense of direction sign ↗ to the stop.
- 4.3 Remove the old carbon brushes. Put the new carbon brushes 20. Make sure that the positioning pin 21 has entered into the hole 22. Do not twist the positioning pin.
The copper-wire of the carbon must stand freely.
- 4.5 Use finger to turn withhold spring in direction of arrow ↖ until stop. Connect cable plug (watch position).
- 4.6 Close cap 18 and rescrew it. Be careful not to twist the cable.

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5. Connecting of adapter ZWA and ZWT for contra-angles respectively hand pieces.

- 5.1 Open chuck (see 2.1).
- 5.2 Unscrew completely coupling ring 23 in the sense of direction sign. (use the shaft of a milling tool \varnothing 2,35 mm)
- 5.3 Remove handpiece from motor.
- 5.4 Put on the adapter and tight it until the pin at the motor flange engages the notch. Tight the adapter in the sense of the direction sign while pressing it against the motor.
- 5.5 All commonly used contra-angles respectively hand pieces can be connected.

Failures

If the handpiece is overloaded or blocked, the apparatus is switched off for security reasons. Restart the apparatus in the following way:

SM 78/3 By pushing the main switch (button 10) switch apparatus off and on. The apparatus then is ready again.

SM 78/4K release knee-switch and than press it again, the apparatus is working again.

SM 78/4F release foot lever and than operate it again, the apparatus is working again.

When **overheated**, the thermic switch 24 on the back side of the apparatus cuts the current. When temperature is normal again the pin may be pushed in place. If the pin comes out often, there is a failure in the apparatus.

If capacity of motor is slowing down or if it is running jerkey, please check carbon brushes or change them.

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declare herewith, that the product

S.M. - Installations consisting of

SM Motorhandpiece 4100 connected with
SM Control unit 4200 or 4560 or 4660

is in conformity with the following provisions of Directive:

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