Georg Schick Dental GmbH
Lehenkreuzweg 12
D-88433 Schemmerhofen
Telefon 07356/9500-0
Telefax 07356/9500-95
Internet: <http://www.schick-dental.de>
E-mail: info@schick-dental.de

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Operating Instructions
Schick-C2 - Profi

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We are pleased that you decided to buy a highly developed piece of equipment from SCHICK and would like to wish you every success when working with your new C2 motor unit.

We wrote these operating instructions to enable you to get accustomed to your new piece of equipment and to provide you with the correct operating and maintenance instructions.

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Index

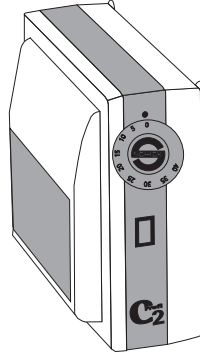
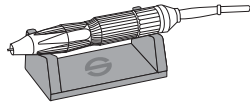
- 1.Summary C2-units
2. List of contents - accessories
3. Range of applications
4. General information
- Caution !**
5. Setting up / operating the unit
6. Exchanging the rotary instruments
7. Replacing the chuck
8. Maintenance and care
9. Possible faults
- 10.Technical data
- 11.Declaration of conformity

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
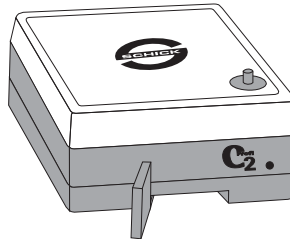
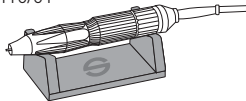
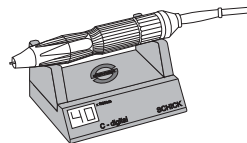
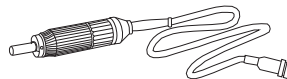
riveting hammer 1
 Burin 1
 twin riveting hammer 2
 double-burin 2

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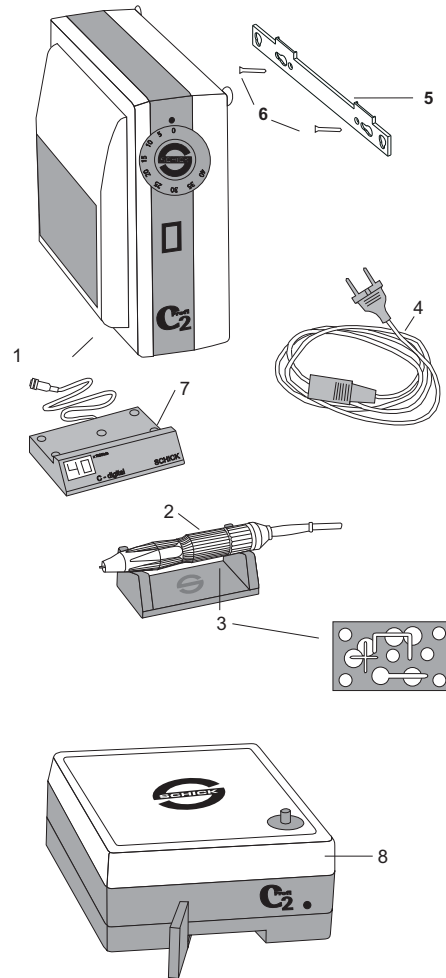2. List of contents - accessories/Spare parts

	Art.-No.
C2/SK motor unit complete	6405/01
C2/SK knee controller 1	6435/01
C2/MH motor-handpiece and cable 2	6200
HA handpiece rest	6385
with two keys for changing the chuck 3	
mains cable 4	5361
suspension strip 5	5614
screws - 2 pieces - 6	3170
C2/SK motor unit complete with digital indicator	6405/11
C2/SK knee controller with attachment for digital indicator 1	6435/11
C2/MH motor-handpiece and cable 2	6200
HA handpiece rest	6385
with 2 keys for changing the chuck 3	
mains cable 4	5361
suspension strip 5	5614
screws -2 pieces - 6	3170
digital indicator 7	6380
C2/SF motor unit complete	6410/01
C2/SF foot controller 8	6440/01
C2/MH motor-handpiece and cable 2	6200
HA handpiece rest	6385
with 2 keys for changing the chuck 3	
mains cable 4	5361
C2/SF motor unit complete with digital indicator	6410/11
C2/SF foot controller with attachment for digital indicator 8	6440/11
C2/MH motor-handpiece and cable 2	6200
HA handpiece rest	6385
with 2 keys for changing the chuck 3	
mains cable 4	5361
digital indicator	6380
ET Chucks:	
Standard: Ø 2.35 mm	4114
Optional Ø 3.0 mm	4117
Ø 3.175 mm	4332
chuck key	5160/01
counter stay wrench	6223
motor cable cpl.	6250

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3. Range of applications

The C2 unit is designed for use in dental laboratories when trimming crowns and bridges, respectively acrylic and chrome cobalt dentures.

The patented SCHICK - C2 motor unit is constructed (intelligent, CPU-controlled, robust, no commutator) so as to run smoothly, thus eliminating user-fatigue and making it economical to use over long periods of time. The processor controller constantly monitors the unit for overloading. A second thermal fuse, which is independent of the computer, provides further protection.

Conditions of environment:

- interior 5° - 40 °
- up to 2,000 meter over sea level

Categorie of overvoltage: II
Grade of pollution: 2

4. General Information

- 4.1 Ascertain that your mains supply coincides with the data on the rating plate.
- 4.2 C2-units are not suitable for the following applications:
 - in areas where there is a risk of explosion
 - on patients
 - for working on moist materials
- 4.3 Ensure that all regulatory requirements are observed during use (always wear protective glasses).
- 4.4 **Under no circumstances should the motor handpiece be cleaned with compressed air.**
- 4.5 Before putting the handpiece down, always insert a rotary instrument or the pin, supplied with the unit, into the chuck.

Caution:

- When using rotary instruments, do not exceed the maximum speeds laid down by their manufacturer.
- Repairs and other technical procedures must only be carried out by suitable qualified personnel, authorized by SCHICK.
- Schick do not guarantee the C2 unit should it not have been used in accordance with the operating instructions.

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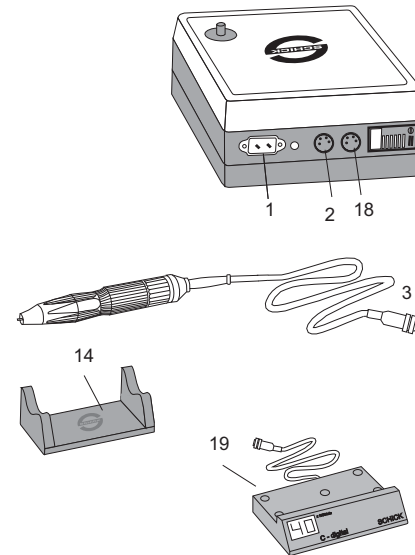
5. Setting up

Operating the knee and foot controller versions

- 5.1 Ascertain that your mains supply coincides with the data on the rating plate.
- 5.2 Plug the mains cable 12 into the unit 1 and into a mains socket.
- 5.3 Plug the motor cable 3 into the socket 2 on the rear of the unit (quarter-turn catch).
- 5.4 Place the handpiece on the rest 14 supplied with it.
- 5.5 Press the mains switch 4 on the front of the unit (for the foot controller version - on the top). The control lamp will come on. The unit is switched off by pressing the mains switch 4 again.

Applies to the knee controller version only

- 5.6 Set the speed controller 5 to the desired speed. The set speed can be reached right away by pressing the knee controller 6.

Applies to the foot controller version only

- 5.7 The speed can be varied progressively between 1.000 - 40.000 rpm by moving the pedal 7.

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Function of Cruising Speed Control - Tempomat

If your new C2 is equipped with a Cruising Speed Control - Tempomat!

Cruising speed control switched on:

- yellow lamp is shining
- each speed which is kept more than 1 sec. is stored
- without pressing the knee-/foot controller the motor is running with the speed stored
- by operating the knee-/foot-control again the stored speed is canceled
- to select a new speed please operate the knee-/foot-control again until the desired speed is reached and keep the control for 1 sec. at this speed; the new speed then is stored.

A stored speed can be canceled by switching off the cruising speed control (yellow lamp "off"). Then there is no operational difference to the standard model.

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Operating the unit with a digital display

Insert the plug on the digital display 19 into the socket 18 in the rear of the unit.

When the unit is switched on, the digital speed indicator shows the pre-selected maximum speed. Once the motor is running, the actual speed of the motor is displayed.

To save space, the digital speed indicator can be placed under the handpiece rest or, if required, adhered to a bench top with double-sided adhesive tape.

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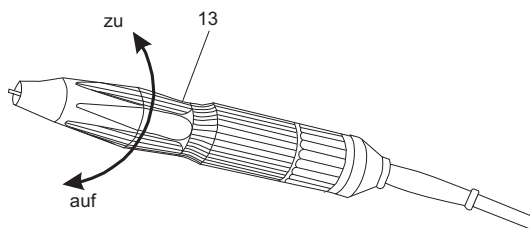
6. Exchanging the rotary instruments

The chuck is opened, respectively, closed by turning the outer sleeve 13 on the handpiece.

Caution:

Only ever exchange the rotary instrument with the **motor switched off!**
With regard to the precision and service-life of the chuck, an instrument must always be inserted into it - even when the handpiece is not in use.
Should the instrument have a short shank, ensure that the head of the instrument does not rub on the end of the handpiece.

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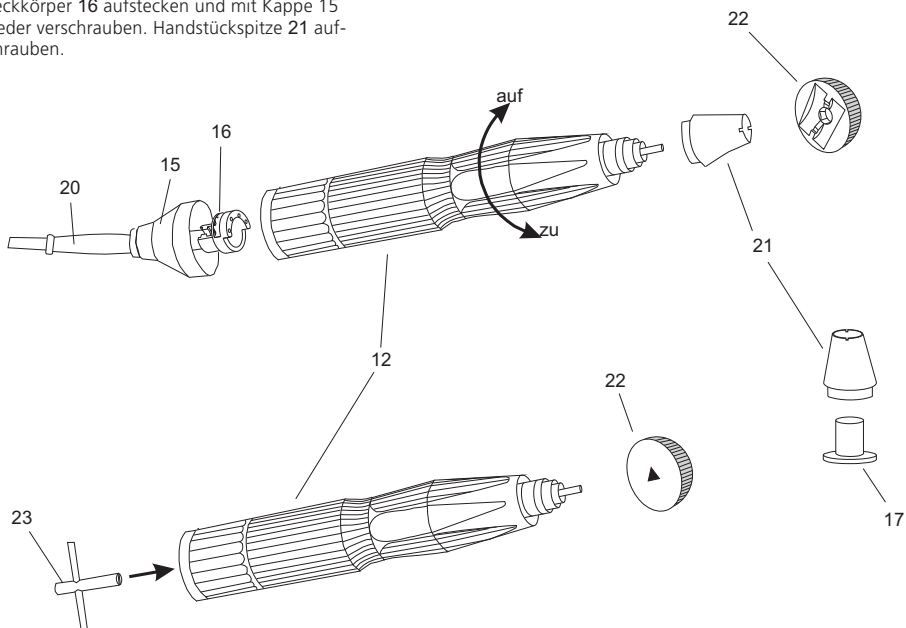
7. Replacing the chuck

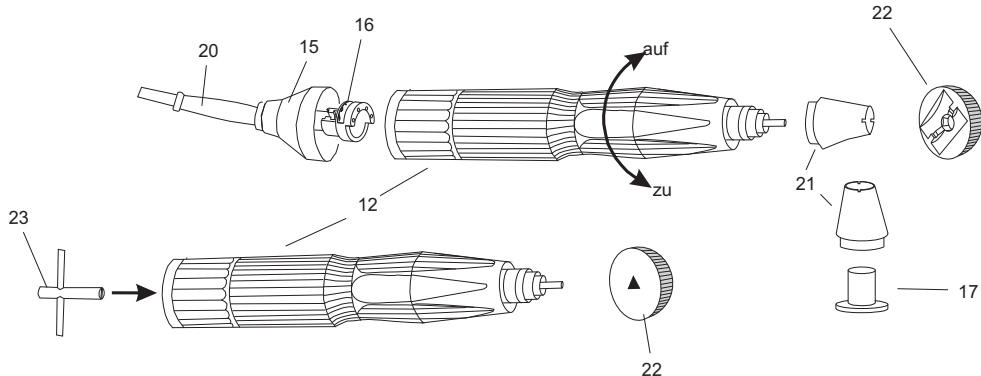
- 7.1 Open the chuck and withdraw the rotary instrument. Unscrew the end of the handpiece **21** (right-handed thread). Should the end be too tight to unscrew, use the milled side of a No. 5160/1 **22** chuck tool by placing its bar in the slot in the end of the handpiece.
- 7.2 Remove the motor cable **20**. To do so, unscrew the cap **15** from the motor and loosen the cable by pulling it out of the plug-in seal **16**. Please ensure that the chuck is open. Use a No. 6223 wrench **23** to hold the motor end of the spindle. Engage the triangular section of the chuck with a No. 5160/1 chuck tool **22**. To prevent the chuck becoming distorted, an instrument shank or the pin supplied must always be inserted into it. Hold the No. 6223 wrench tightly and turn the chuck tool anti-clockwise to unscrew the chuck completely - the chuck has a right-handed thread!
- 7.3 Clean the chuck, grease it lightly and place it in the spindle. Use the tools to screw the chuck in - clockwise and as far as possible - as described in section 7.2, **and tighten it slightly**. Replace the plug-in seal **16** and screw the cap **15** back into place. Screw the end of the handpiece **21** on again.

Please note:

The tools **22** and **23** are to be found in the base of the handpiece rest!

If you intend to replace the standard 2.35 mm chuck with a 3.0 mm version, the bush in the end of the handpiece must be replaced with a new one (article No. 6279). To do so, remove the end of the handpiece, press the bush **17** downwards, out of the tip and place the new bush in position.

When using a special handpiece for left hand (marked with blue ring) the chuck is replaced accordingly, but with left handed thread.

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8. Care and maintenance

Caution: Do not clean your C2 motorhandpiece with compressed air!

The chuck should be cleaned and re-greased once in a while, depending on how dirty it is (refer to section 7).

As C2 units have no commutators, carbon brushes or ventilation parts, no further maintenance is required.

9. Possible faults

Should the handpiece be overloaded, respectively, jammed, for safety reasons the unit will switch itself off.

- When using a foot or knee controller, release the pedal switch, respectively knee switch and operate it again; the unit is ready for use immediately.

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10. Technical Data

	C2/SK	C2/SF
Dimensions:	Width:	120mm 230mm
	Height:	230mm 110mm
	Depth:	245mm 245mm
Weight:	3,3 Kg	3,3Kg
C2-MH	Length: 180mm	Weight 240g
Power supply:	AC 220-240V 115V 100V	
Nominal frequency:	50/60Hz	
Power input:	170W	
Type:	C2	
Handpiece model:	6xx.xxx	
Speed range:	1.000 - 40.000 rpm	
Safety class	II	

This unit complies with the current VDE (German association of electrical technicians) regulations concerning safety and suppression. The C2 is a group III unit according to Med. GV type B, and does not include an application component.

These operating instructions should be readily accessible and are best kept close to the unit itself.

We would like to take this opportunity to advise you that a proper repair service and suitable qualified personnel are required for such highly developed technical equipment.

SCHICK guarantees to carry out perfect repairs using original spare parts.

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Nous, GEORG SCHICK DENTAL GmbH