**SCHICK**  
Dentalgeräte

Georg Schick Dental GmbH  
Lehenkreuz 12  
D-88433 Schemmerhofen  
Telefon 07356/9500-0  
Telefax 07356/9500-95  
Internet: <http://www.schick-dental.de>  
E-mail: [info@schick-dental.de](mailto:info@schick-dental.de)

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## Operating Instructions Schick-C3 Master

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**We are pleased that you decided to buy a highly developed piece of equipment from SCHICK and would like to wish you every success when working with your new C3 motor unit.**

**We wrote these operating instructions to enable you to get accustomed to your new piece of equipment and to provide you with the correct operating and maintenance instructions.**

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## **Index**

1. Summary C3-units
2. List of contents - accessories
3. Range of applications
4. Safety informations
- Caution !**
5. Setting up / operating the unit
6. Cruising System/digital display
7. Exchanging the rotary instruments
8. Replacing the chuck
9. Maintenance and care
10. Possible faults
11. Technical data
12. Declaration of conformity

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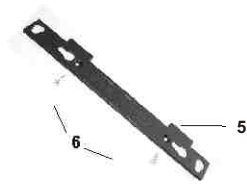
riveting hammer 1  
burin 1  
twin riveting hammer 2  
double-burin 2

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7103  
 7102  
 5160/1  
 5158  
 6223  
 7250

## 2. List of contents - accessories/Spare parts

<b>C3 motor unit type SK complete</b>	Art.-No.
knee controller <b>1</b>	<b>7005</b>
motor-handpiece and cable <b>2</b>	<b>7035</b>
handpiece rest - with three keys for changing the chuck- <b>3</b>	<b>7000</b>
mains cable <b>4</b>	<b>6385</b>
suspension strip <b>5</b>	<b>2160</b>
screws - 2 pieces - <b>6</b>	<b>5614</b>
	<b>3170</b>

<b>C3 motor unit type SK complete with digital indicator</b>	<b>7005/1</b>
knee controller with attachment for digital indicator <b>1</b>	<b>7035/1</b>
motor-handpiece and cable <b>2</b>	<b>7000</b>
handpiece rest - with three keys for changing the chuck- <b>3</b>	<b>6385</b>
mains cable <b>4</b>	<b>2160</b>
suspension strip <b>5</b>	<b>5614</b>
screws - 2 pieces - <b>6</b>	<b>3170</b>
digital indicator <b>7</b>	<b>6380</b>

<b>C3 motor unit type SF complete</b>	Art.-No.
foot controller <b>8</b>	<b>7010</b>
motor-handpiece and cable <b>2</b>	<b>7040</b>
handpiece rest - with three keys for changing the chuck- <b>3</b>	<b>7000</b>
mains cable <b>4</b>	<b>6385</b>
	<b>2160</b>

<b>C3 motor unit type SF complete with digital indicator</b>	<b>7010/1</b>
foot controller with attachment for digital indicator <b>8</b>	<b>7040/1</b>
motor-handpiece and cable <b>2</b>	<b>7000</b>
handpiece rest - with three keys for changing the chuck- <b>3</b>	<b>6385</b>
mains cable <b>4</b>	<b>2160</b>
digital indicator <b>7</b>	<b>6380</b>

<b>C3 motor unit type ST complete</b>	Art.-No.
control unit table top model <b>9</b>	<b>7015</b>
motor-handpiece and cable <b>2</b>	<b>7045</b>
dynamic foot-switch <b>10</b>	<b>7000</b>
handpiece rest - with three keys for changing the chuck <b>3</b>	<b>6371</b>
mains cable <b>4</b>	<b>6385</b>
	<b>2160</b>

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### 3. Range of applications

The C3 unit is designed for universal use in dental laboratories when trimming crowns and bridges, respectively acrylic and chrome cobalt dentures.

With its speed range from 1.000 - 50.000 rpm the SCHICK - C3- unit allows to work all dental materials. For your safety a mechanical locking is fitted into the speed controller.

The motorhandpiece is ergonomical, small shaped and runs smoothly, thus eliminating user-fatigue and making it economical to use over long periods. The progressive processor controller constantly monitors the unit for overloading. A second thermal fuse, which is independent of the computer, provides further protection. Thus the C3-unit offers a maximum of safety.

Conditions of environment:

- interior 5° - 40 ° C.
- up to 2,000 meter over sea level

Categorie of overvoltage: II  
Grade of pollution: 2

### 4. Safety Informations

- 4.1 Ascertain that your mains supply coincides with the data on the rating plate.
- 4.2 C3-units are not suitable for the following applications:
  - in areas where there is a risk of explosion
  - for medical applications
  - for working on moist materials
- 4.3 Ensure that all regulatory requirements are observed during use (always wear protective glasses).
- 4.4 **Under no circumstances should the motor handpiece be cleaned with compressed air.**
- 4.5 Before putting the handpiece down, always insert a rotary instrument or the pin, supplied with the unit, into the chuck.

#### Caution:

- When using rotary instruments, do not exceed the maximum speeds laid down by their manufacturer.
- When operating with left-hand rotation, the collet may loosen when there is a high load.
- Repairs and other technical procedures must only be carried out by suitable qualified personnel, authorized by SCHICK.
- Schick do not guarantee the C3 unit should it not have been used in accordance with the operating instructions.

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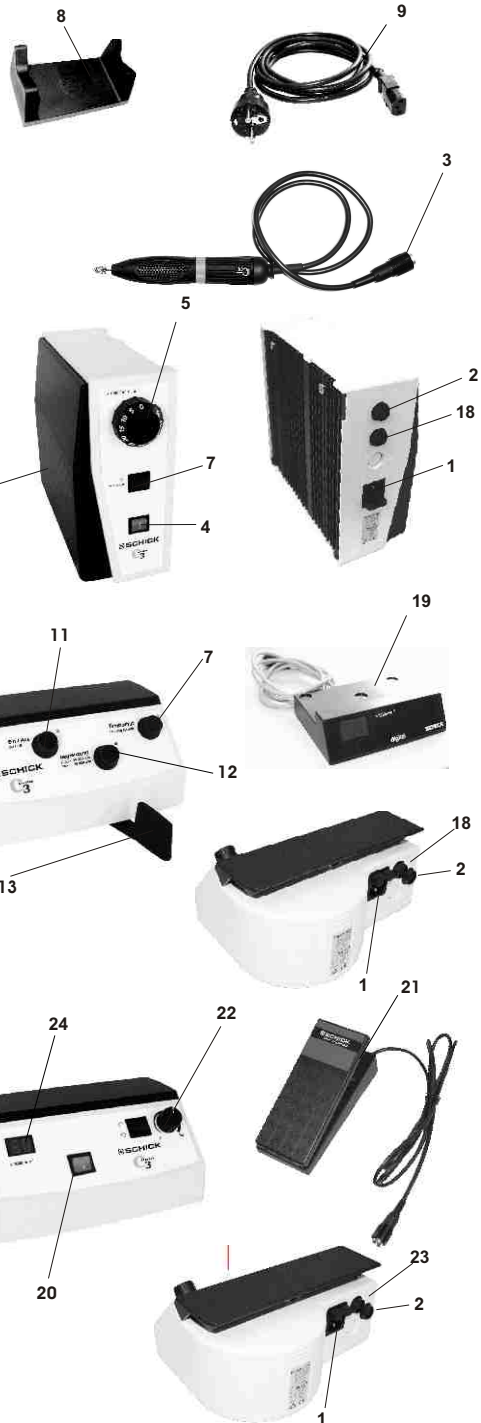
## 5. Setting up

### Operating the different versions

Cruising system and operating with digital indicator see description on page 11.

#### all versions

- 5.1 Ascertain that your mains supply coincides with the data on the rating plate.
- 5.2 Plug the mains cable **9** into the unit **1** and into a mains socket.
- 5.3 Plug the motor cable **3** into the socket **2** on the rear of the unit (quarter-turn catch).
- 5.4 Place the handpiece on the rest **8** supplied with it.

#### knee controller version only

- 5.5 Press the mains switch **4** on the front of the unit. The control lamp will come on. The unit is switched off by pressing the mains switch **4** again.
- 5.6 Set the speed controller **5** to the desired speed. The set speed can be reached right away by pressing the knee controller **6**. The speed controller is provided with a mechanical locking which only allows speeds up to 40.000 rpm unless the segment **5a** is operated deliberately. To reach speeds > 40.000 rpm press segment **5a** whilst turning speed controller

#### foot controller version only.

- 5.5 Press the mains switch **11** on the front of the unit. The LED will flash up. The unit is switched off by pressing the mains switch **11** again.
- 5.6 The speed can be varied progressively between 1.000 - 50.000 rpm by moving the foot pedal **13**. For your safety the unit is provided with a mechanical speed limitation **12**. If the red warning light lights up, up to 50.000 rpm have been enabled. If the red warning light at the limiting switch is not lit up, the RPM's are limited to a maximum of 30.000.

#### bench controller version only

- 5.5 Plug the dynamic foot-switch **21** into the socket **23** on the rear of the unit (quarter-turn-catch).
- 5.6 Switch on the mains switch **20** on the front of the unit (the control lamp lights up). The unit is switched off by pressing the mains switch **20** again.
- 5.7 Set the speed control knob **22** to the desired speed. (see digital display **24**) The motor can be switched on or off by moving the pedal and the speed can be varied progressively up to the set speed.
- 5.8 The bench unit can also be operated without the foot-switch. Set the speed control knob **22** to the desired speed, the motor will begin to run immediately.

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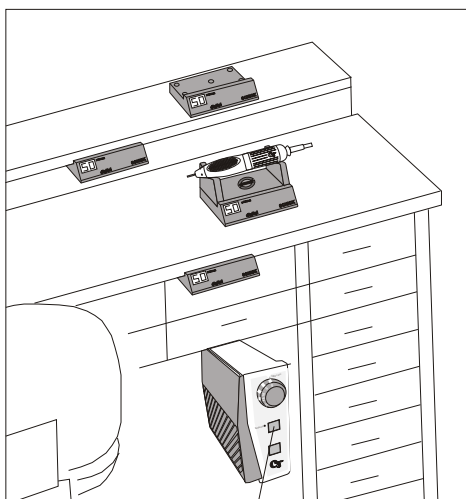
## 6. Function of Cruising Speed Control - Tempomat

Your **C3 knee/foot-controller** is equipped with a **cruising speed control -Tempomat-**. This extended control system enables the operator to select and store a given speed. A considerable facility for maintaining a constant speed f. e. for work underneath the microscope or for extensive work.

Cruising speed control **7** switched 'on'  
- yellow LED is shining  
- each speed which is kept more than 1 sec. is stored  
- without pressing the knee/foot controller the motor is running with the speed stored  
- by operating the knee/foot controller again or by pressing the switch **7** the stored speed is canceled  
- to select a new speed please operate the knee/foot control again until the desired speed is reached and keep the control for about 1 sec. at this speed; the new speed then is stored.

A stored speed can be canceled by switching off the cruising speed control (yellow LED not shining).

## Operating the unit with a digital display

### knee and foot models only

Insert the plug on the digital display **19** into the socket **18** in the rear of the unit. When the unit is switched on, the digital speed indicator shows the pre-selected maximum speed. Once the motor is running, the actual speed of the motor is displayed.

To save space, the digital speed indicator can be placed under the handpiece rest or, if required, adhered to a bench top with double-sided adhesive tape.

### table top unit

into the table top unit a digital display is integrated - the display operates in the same way as described above

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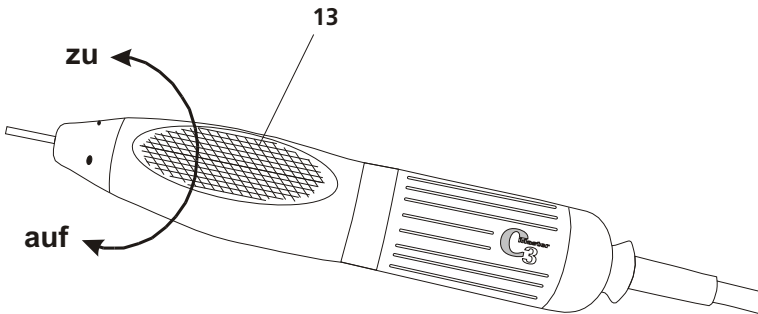
## 7. Exchanging the rotary instruments

The chuck is opened, respectively, closed by turning the outer sleeve **13** of the handpiece.

### Caution:

Only ever exchange the rotary instrument with the **motor switched off!**

With regard to the precision and service-life of the chuck, an instrument must always be inserted into it - even when the handpiece is not in use.

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
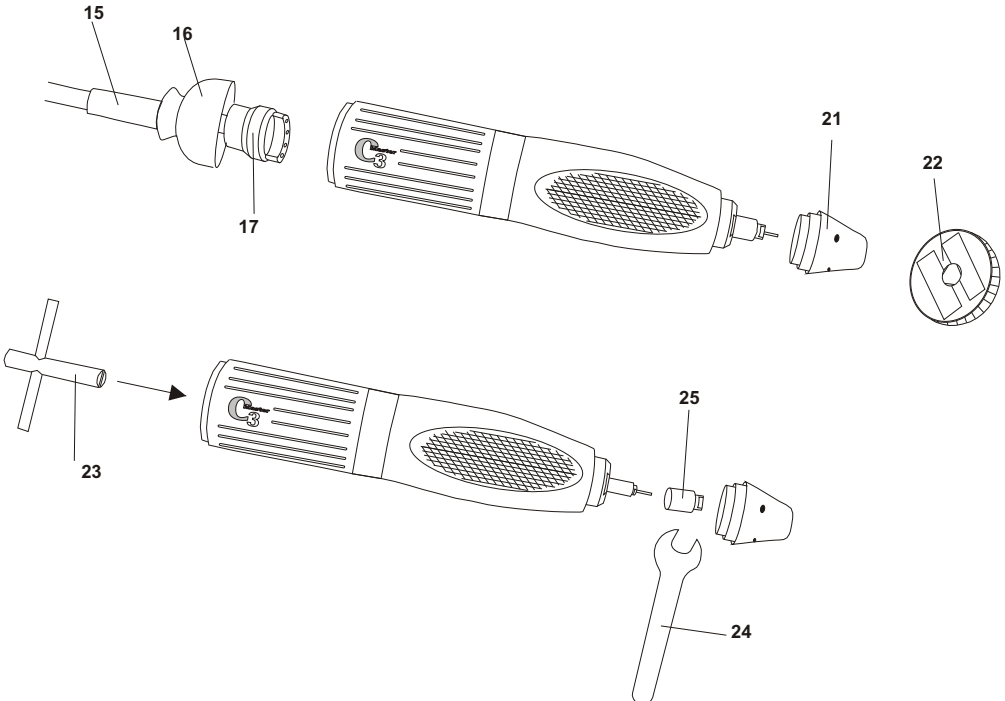
## 8. Replacing the chuck

- 8.1 Open the chuck and withdraw the rotary instrument. Unscrew the end of the handpiece **21** (right-handed thread). Should the end be too tight to unscrew, use the milled side of a No. 5160/1 **22** chuck tool by placing its bar in the slot in the end of the handpiece.
- 8.2 Remove the motor cable **15**. To do so unscrew the cap **16** from the motor and loosen the cable by pulling out the plug-in seal **17**. Please insure that the chuck is open.
- 8.3 Use a number 6223 wrench **23** to hold the motor end of the spindle. Unscrew sealing cap **25** (right-handed thread) with wrench 5158 **24**.
- 8.4 Engage the triangular section of the chuck with a No. 5160/1 chuck tool **22**. To prevent the chuck becoming distorted, an instrument shank or the pin supplied must always be inserted into it. Hold the number 6223 **23** wrench tightly and turn the chuck tool anti-clockwise to unscrew the chuck completely - the chuck has a right-handed thread!
- 8.5 Clean the chuck, grease it lightly and place it in the spindle. Use the tools to screw the chuck in - clockwise and as far as possible - as described in section 8.3 and 8.4, **and tighten it slightly**. Replace the plug-in seal **17** and screw the cap **16** back into place. Screw the sealing cap **25** and the end of the handpiece **21** on again.

### Please note:

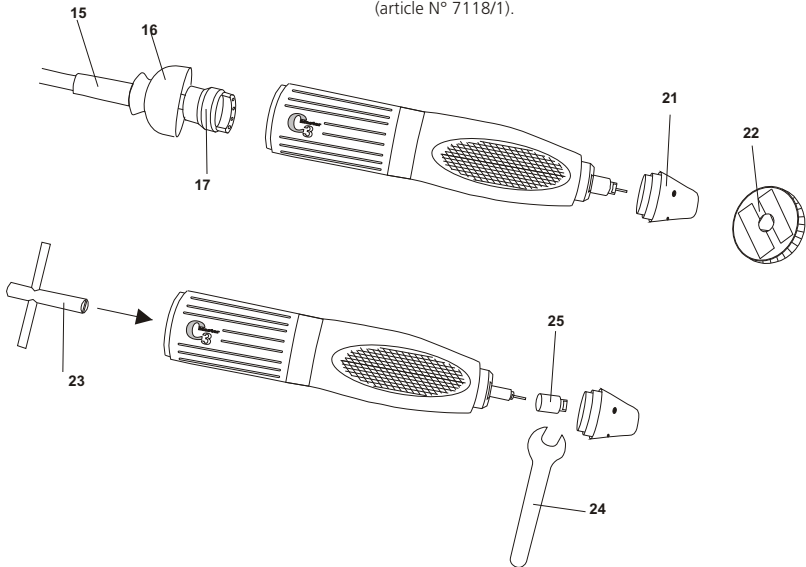The tools **22**, **23** and **24** are to be found in the base of the handpiece rest!  
If you intend to replace the standard 2,35 mm chuck with a 3.0 mm version, the sealing cap **25** must be replaced with a new one (article no. 7118/1).

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## 9. Care and maintenance

**Caution: Do not clean your C3 motorhandpiece with compressed air !**

**Chuck:** should be cleaned and re-greased once in a while, depending on how dirty it is.

**O-Ring** placed inside of sealing cap: grease slightly from time to time using a greased toolshaft.

As C3 units have no commutators, carbon brushes or ventilation parts, no further maintenance is required.

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## 10. Possible faults

Should the handpiece be overloaded, respectively, jammed, for safety reasons the unit will switch itself off.

- Release knee/foot switch and operate it again; the unit is ready for use immediately

**bench controller version only**

- When using the bench version without a foot switch, the speed control knob should be turned to "0", when the desired speed is selected again; the unit is ready for use immediately.
- When using the bench version with a foot switch it should be released and operated again; the unit is then ready for use again.

**all versions**

if the unit should not be running after doing as described above the 2 fuses have to be checked and eventually exchanged. The fuses (2 x 2A inert) are to be found at the rear of the unit beneath the socket **1**.

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This unit complies with the current VDE (German association of electrical technicians) regulations concerning safety and suppression.

These operating instructions should be readily accessible and are best kept close to the unit itself.

We would like to take this opportunity to advise you that a proper repair service and suitable qualified personnel are required for such highly developed technical equipment.

SCHICK guarantees to carry out perfect repairs using **original spare parts**.

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Nous, GEORG SCHICK DENTAL GmbH