

## Operating Instructions riveting hammer QUBE



NH QUBE  
art.-no. 1850/2

The riveting hammer QUBE is individually used for various works in dental labs. The riveting hammers QUBE are especially suitable to detach made up telescope- and conuscrowns without any damage of the sensitive periphery of the primary – and secondary part. Also loosening broken off anchors can easily be done with this riveting hammer. Another field of application is the riveting itself. The riveting hammer QUBE can be used with all SCHICK-QUBE handpieces.

To remove plaster or investement the chisel (pic.1) is used. With the hexagonal guide of the chisel the final position of the chisel's tip can be defined in 3 different positions.



### General informations:

The riveting hammer QUBE must be used with **max. 15.000rpm** and is not suitable for use on patients. During use all regulatory requirements have to be observed (always wear protective glasses!)

**Under no circumstances should the riveting hammer QUBE be cleaned with compressed air!** Schick do not guarantee the riveting hammer QUBE should it not have been used in accordance with the operating instructions.

### Setting up

Ascertain that the chuck's diameter in the used motorhandpiece is  $\varnothing 2,35\text{mm}$ . Open chuck and remove the tool. Put riveting hammer QUBE on by placing the shaft ( $\varnothing 2,35\text{mm}$ ) into the chuck.

**Caution:** When closing chuck mechanism take care that the riveting hammer QUBE is placed very close to the tip of the handpiece – so it's secured against rotation.

Maintenance and care

**Caution: Do not clean riveting hammer QUBE with compressed air!** The riveting hammer QUBE is equipped with closed and permanent lubricate ball bearings – therefore no regular maintenance is necessary. If the riveting hammer QUBE becomes more and more ineffective, the striker should be cleaned and greased. To clean / exchange striker or chisel the guidance sleeve has to be detached from housing by using an engineer's wrench SW6 (pic.1). Remove striker / chisel from sleeve. Clean striker / chisel with a soft cloth and grease slightly. Re-assembling viceversa.

**Caution:** Thread has to be secured with "Loctite liquid adhesive" (our ref. No. W7030000006).

Chisel with sleeve (ref.-no. 1855)

**pic.1**



sleeve for chisel  
art.-no. 1862/1

sleeve for striker  
art.no. 1862



chisel  
art.no. 1863/1

striker  
art.no. 1863



housing  
riveting hammer  
QUBE  
art.no. 1861/2



riveting hammer  
QUBE  
with chisel  
art.no. 1850/3

SCHICK guarantees to carry out perfect repairs using original spare parts.



Subject to technical modification without prior notice



Schick GmbH  
Lehenkreuzweg 12  
D-88433 Schemmerhofen  
Telefon +49 7356 9500-0  
Telefax +49 7356 9500-95  
E-Mail info@schick-dental.de  
Internet www.schick-dental.de