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All our instruments are carefully developed and manufactured for their respective tasks. Improper usage may endanger the user and the patient and possibly also third parties, and may damage or destroy the instrument or the hand piece.

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Precautions before use:

1. Make sure that only technically and flawless hand and right angle pieces are used.
2. Please clamp in the shaft of the instrument as deeply as possible. Instruments that are loose or protrude too far may detach or break and cause injury.
3. Please select the speed so that the maximum permissible can not be exceeded (see «Maximum RPM»). Exceeding the maximum speed increases the safety risk, reduces the quality of work and generates heat.
4. Do not continue using blunt or damaged instruments! Please check the instruments before each use! In diamond tools, danger signals include bare spots on the working piece, warped instruments that cause vibrations at start-up and changes of the original shape. Carbide cutting tools show damaged and deformed blades or breaks.
5. Please bring the instrument to working speed before contact with the work piece, respectively. If any vibrations occur, the instrument is warped and therefore may not be used anymore!
6. Please follow the instructions of the hand piece manufacturer!

Failure to comply with the safety advice may lead to damage to the work piece, and may pose a danger to user, or third parties.

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