# Deutsch

Schick GmbH  
Lehenkreuzweg 12  
D-88433 Schemmerhofen  
Telefon +49 7356 9500-0  
Telefax +49 7356 9500-95  
E-Mail [info@schick-dental.de](mailto:info@schick-dental.de)  
Internet [www.schick-dental.de](http://www.schick-dental.de)

Original

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## Instructions Turbine T100

E

We are pleased that you decided to buy a highly developed piece of equipment from SCHICK and would like to wish you every success for working with your new turbine.

We wrote these instructions to enable you to get accustomed to your new equipment and to provide you with the correct operating and maintenance instructions.

## Index

1. Range of applications
2. Safety information
3. List of contents
4. Setting up
5. Handling
6. Guarantee
7. Technical data
8. Maintenance and Care
9. Optional accessories
10. Declaration of Conformity

## 1. Range of applications

The T100 is a high performance turbine which is adapted to the technical needs of working on full ceramics.

The T100 can be used in all common milling-machines by using an adapter. The unique LED illumination (optional) supports the fabrication of high precision dental restorations. The turbine is not suitable for clinical applications.

## 2. Safety information

The device must be used by trained staff in accordance with the current legal provisions concerning industrial safety, health and accident prevention regulations and these working instructions. In accordance with these requirements, the operator:

- must only use working devices that are in perfect working order. In the event of irregular functioning, excessive vibration, not normal heating or other things indicating malfunction of the device, the work must be stopped immediately. In this case, contact a repair centre that is approved by Schick Dental.
- must ensure that the device is used only for the purpose for which it is intended.

The device is not intended for medical treatment. Any use other than that for which this product is intended, is not allowed and may be dangerous. The device meets the current European legal requirements.

The Turbine is not suitable for the following field of applications:

- in areas with risk of explosions
- for medical applications

## Recommendations:

It is essential to use dry, purified compressed air in order to ensure long working life of the device. Maintain air quality and water by regular maintenance of the compressor and the filtration systems. The use of unfiltered hard water will lead to early blockage of the tubes, connectors and spray cones.

After a period of prolonged use, the device must be cleaned and serviced according to these instructions.

## 3. List of contents

- Turbinhandpiece  
including grease, bur inserting tool and spray-needle
- Control unit  
with integrated water-tank, ca. 1400 ml
- Foot switch
- All tube parts,  
including connections
- Beaker
- Funnel

## 4. Setting up

Attach the control unit in accordance to the picture.

- Push the lock nuts onto the tubes of the foot switch.  
Attach the foot switch tubes to the corresponding terminal and fix them with the locknuts.  
(left: dark grey hose, right: light grey hose)

- Attach the compressed air supply to the control unit by putting the compressed air hose into the corresponding connection.

Attention: - Compressed air tube must click in perceptibly!  
- Please take care that the compressed air connection is first connected to the milling unit and then to the compressor!

## 5. Handling

### 5.1 Refilling of water tank

Unscrew lid slowly to release pressure. Refill tank with distilled water by using the accessories. Fill the tank, until the water reaches the upper end of the sight glass (approx. 1,4 liter). Fasten the lid slightly. The pressure inside the tank increases.

### 5.2. Spray activation and adjustment

Activate spray by switching the lever into the ON position. Water amount can be adjusted with the green button and air amount with the blue button.

### 5.3 Operation

Choose Spray OFF / ON, activate foot control to start the turbine and depending on the switch position the water spray starts also.

### 5.4 Inserting the tool

Place the tool inside the borehole. Push the turbine head inside the inserting tool until it snaps in audibly.

### 5.5 Extracting the tool

Hold the middle part of the turbine with one hand and push the tail forwards until the stop. The tool can be taken out of the chuck.

### 5.6 Pressure regulator with valve

The Turbine is equipped with an automatic ventilation system. We recommend to release the pressure in the tank once in a week to activate the ventilation.

## 6. Terms of warranty

Schick Dental grants the operator a guarantee covering all functional defects, material or production faults. The device is covered by this warranty for 12 months from the date of invoice.

In the case of justified claims, Schick Dental or it's authorised representative will fulfill the company's obligations under this warranty by repairing or replacing the product free of charge.

Any other claims in particular in the form of damages and interest, are excluded.

Schick Dental shall not be held responsible for damage or injury and their consequences resulting from:

- excessive wear and tear
- improper handling
- non-observance of the instructions for installation, operation and maintenance
- unusual chemical, electrical or electrolytic influences.
- poor connections, whether of the air, water or electricity supply

The warranty becomes void if the damage and its consequences are due to improper manipulation of the product, or modifications to the product carried out by persons not authorized by Schick Dental.

Claims under the terms of warranty will be considered only on presentation, of the product together with the invoice or the delivery note on which the date of purchase, the product reference and the Serial No.

## 7. Technical Data

### Classification

In accordance with European Directive 2006/42/EG regarding machinery.

Dimensions:   Width: 295 mm  
                  Height: 192 mm  
                  Depth: 155 mm

Weight:           3.200 g  
Tank volume:   1,4 litres

WEEE-Reg.-Nr. DE 78620387

Intake air pressure: Minimum 5 bar and maximum 8 bar

The station is delivered with a working pressure adjusted at 3 bar

Please rely on the station manometers data.  
We suggest an air pressure of 3 bar.

Transport and store conditions:   Temperature range -40°C to 70°C  
  Relative humidity between 10% und 100%  
  Atmospheric pressure between 0,5 bar and 1,06 bar  
  Up to 2000 m NN

Burs: Shaft diameter 1,60 mm according to ISO 1797 Typ 3, Length max. 25 mm,  
      Code 4-6 according ISO 6360-1

turbine handpiece:	Recommended working speed:	Max. 300.000 U/min
	Weight:	60 g
	Air consumption:	35 litres / min. approx.
	Spray-air consumption:	2 litres / min. at 2 bar
	Spray-water consumption:	100 ml/ min. at 2 bar
	Noise level:	According to ISO 7785-1, at 45 cm:           70 dB

## 8. Maintenance and Care

Only use maintenance products and components from Schick Dental. The use of other products and components can lead to annulment of the warranty.

Never disassemble the device. For all modifications and repairs, we recommend you to contact your regular supplier or Schick Dental directly. Schick Dental advises the user to have its dynamic instruments checked or inspected at least once a year.

- Clean housing with water and a soft cleaning agent only
- Never dip into disinfectant
- Never dip into ultrasonic

Lubrication:

Lubrication should be done at least twice a day. Only use Lubrimed medical grease and greaser:

- First-time use: Remove the cap of the greaser, screw the knurled back-housing until the grease appears at the end of the rod.
- Remove the bur and introduce the rod of the greaser to the bottom. The dose corresponds to one half-turn of the knurled back housing.
- Replace a bur, turn on the air and run the turbine for about 15 seconds
- Wipe off surplus grease and replace cap after use.

Cleaning of spray tubes with the wire cleaner!

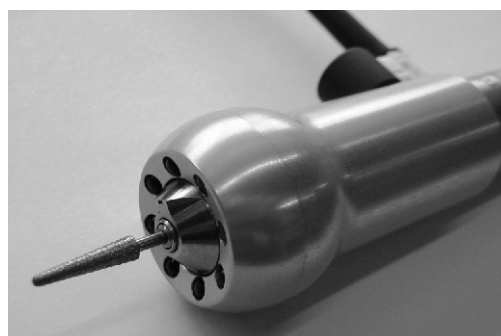
Important: After cleaning and disinfecting the instrument and before using it, operate at moderate speed, with a bur in the chuck for 10 to 15 seconds to distribute, and remove the excess lubricant!

Important: The device must not be started without a tool inserted into the chuck!

## 9. Optional Accessories

It is possible to equip your turbine with a light head.

The light head illuminates the working area optimal. Light, exactly where it is needed. The light intensity can be adjusted by a potentiometer variable.

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declare herewith, that the product

Turbine T 100 art.-no. 2640/1

is in conformity with the following provisions of Directive:

2006/42/EG (machinery directive)

Name and address of person in charge: Wolfgang Schick  
Lehenkreuzweg 12  
88433 Schemmerhofen

Schemmerhofen, February 2016

W. Schick  
director

Manufacturer:

Schick GmbH  
Lehenkreuzweg 12  
D-88433 Schemmerhofen  
Telefon +49 7356 9500-0  
Telefax +49 7356 9500-95  
E-Mail [info@schick-dental.de](mailto:info@schick-dental.de)  
Internet [www.schick-dental.de](http://www.schick-dental.de)

Subject to technical modification without prior notice

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We, SCHICK GmbH